

HYGENICA IPC CURTAINS

PRODUCT CARE AND STORAGE

Usage and Care Instructions

Product that becomes visibly soiled or stained during use should be replaced. The product is a single use disposable item, and must not be laundered or re-used. Cleaning agents should not be applied as this will affect the performance of the product.

Maintenance

The curtain requires little to no maintenance during normal usage. It is recommended that the curtain be placed in the curtain tidy when not in use. The product will benefit from light lubrication of the tracking every 4-6 weeks. This reduces friction and enhances the ease of use of the curtain. A silicone based lubricant applied using an impregnated wipe is recommended, and these can be supplied by Hygenica.

Storage

Curtains should be stored boxed, out of direct sunlight and in conditions between 10°C to 40°C at less than 80% humidity.

Curtain Replacement

- Fold the curtain back into the plastic curtain tidy. • Remove the clip or end stop from the tracking access point.
- Slide curtain out from track.
- Lubricate the track using an AiOM silicone wipe.
- Place new curtain into the track. Keep plastic sleeve on the curtain during replacement.
- Replace clip into tracking access point, leaving a final hook on the wall side of the clip if desired.
- Add installation date onto the curtain.

